




**AMSX 2572 & AMSX 3076
LAMINATE CHANGING TABLE**

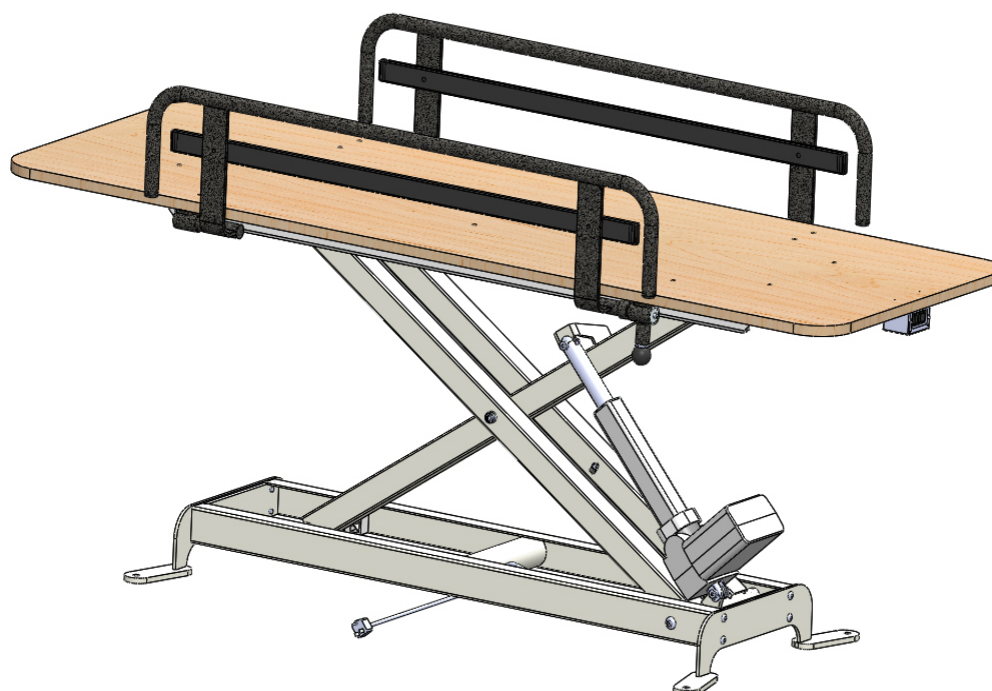


PARTS LIST

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	18856	BASE FRAME SX PNTD	1
2	18962	SX CSTR LEG S/A 7IN CLRNC PNTD	2
3	18775	ARM SX-BA DUAL LIFT PNTD	1
4	18777	ARM AM-SX 2x2 PWR SINGLE LIFT PNTD	1
5	18998	BRKT SIDE RAIL PVT LOCK MNT LF PNTD	1
6	18995	BRKT SIDE RAIL PIVOT MOUNT LR PNTD	1
7	18980	BRKT SIDE RAIL PVIOT MNT RR PNTD	1
8	18982	BRKT SIDE RAIL PVT LOCK MNT RF PNTD	1
9	18991	FRAME SX SIDE RAIL LEFT PNTD	1
10	18993	FRAME SX SIDE RAIL RIGHT PNTD	1
11	28349	BOX SWITCH AMSX2572 ASSEMBLED	1
12	14851	BD LAMINATE SX CHG TBL 25x72	1
13	13854	BAR RD 1/2x8-1/8 SX PVT PLTD	1
14	13879	BUSH SX BRZ .5015x.628x1/4" OILITE	4
15	11853	TUBE 1/2x1.5x16GAx40 RAIL GUARD PNTD	2
16	02198	BTTN HD SOCKET SCREW 1/2-13x3/4	4
17	02261	FRM SX CHANGING TABLE TOP PNTD	1
18	02012	BOLT SHLDR M10x30	2
19	02042	SCREW BTN HD SOC 1/4-20x1/2	2
20	02045	SCREW HX HD CAP 1/4-20x1	14
21	02092	NUT NYLOCK M8-1.25	2
22	02165	WASHER NYLON 1/2 ID	2
23	02176	SCREW BTN HD SOC 3/8-16X3/4	8
24	02196	CAP AXEL NUT 1/2"	2
25	02200	CLAMP CABLE 3/16 BLACK	4
26	02204	CABLE TIE ADHESIVE MOUNT	2
27	02235	CABLE TIE 1/16" X 6"	2
28	02280	WASHER FENDER 1/4x1-1/4OD SS	2
29	02340	SX SLIDE BLOCK PLAS	4
30	02507	SCREW SHT MTL TRUSS HD PHIL #8x3/4	2
31	02542	SCREW HX WSHR HD #10 x 3/4 SELF TAP	4
32	02657	PLUG RD 1"ODx14-20 GA BLK	4
33	02715	SCREW SET #12-24x1/2 SOC HD	8
34	02716	PLUG 1/2x1-1/2 BLACK	4
35	03605	MOTOR MAX74 8KN 120-240VAC W/GROUND	1
36	03607	CABLE AC MAINS W/GRND MAX7	1
37	03126-ASY	SPRING PIN PULL BALL	2
38	14865	BD LAMINATE SX CHG TBL 30x76	1

AMSX 2572 & AMSX 3076

TOP

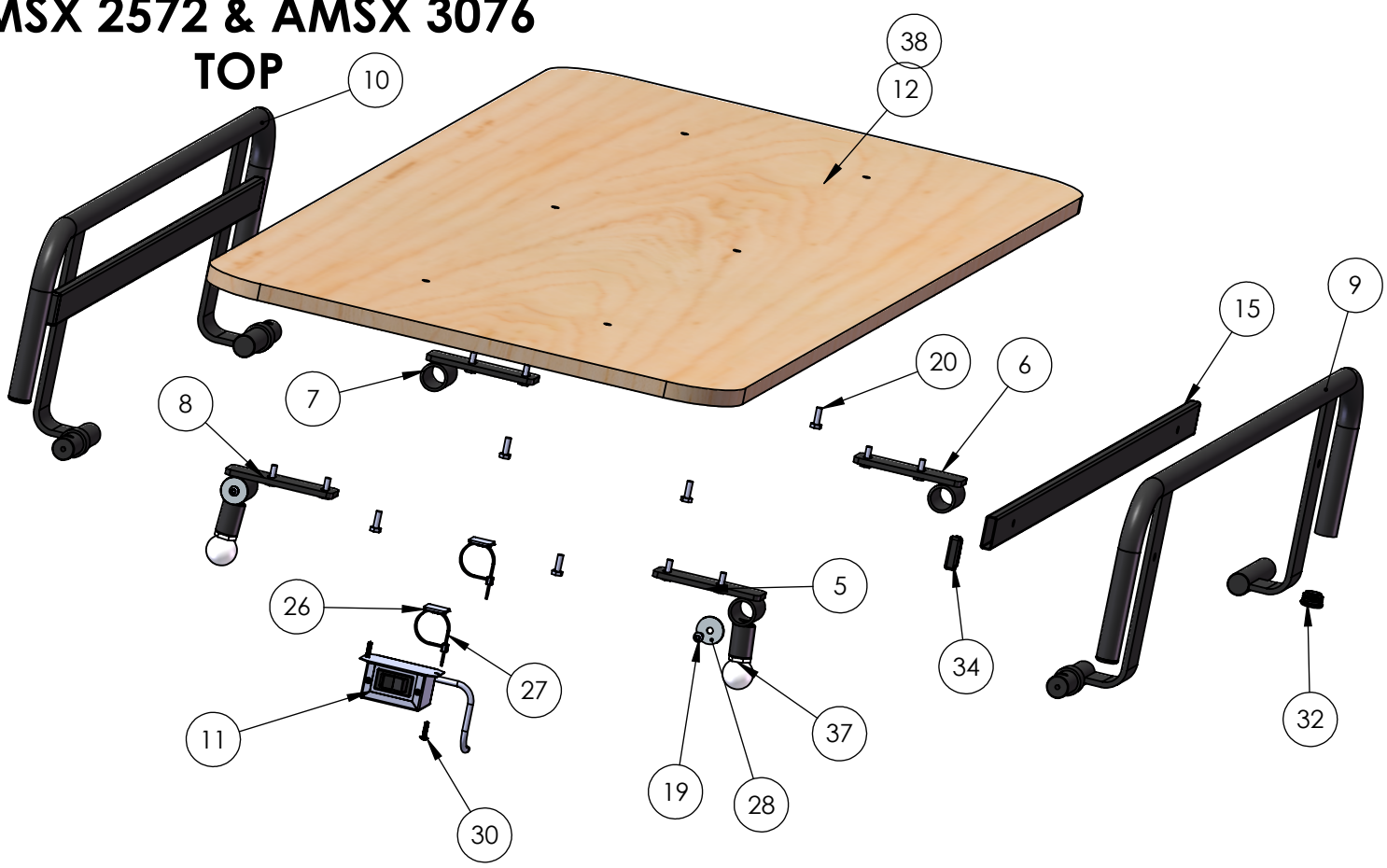

AMSX 2572 & AMSX 3076

BASE

