



Hands Free Electric Table

(Basic, Lift Back, Deluxe versions)

User Manual

Applies to the following Models:

PT9020 (115V)

PT9021 (220V)

PT9026 (115V)

PT9027 (220V)



Product Description

Intended Use: Hands Free Electric Massage Tables are intended for use in a clinical environment to support patients during massage or spa treatments. This table should be used only by trained personnel.

<u>Model No.</u>	<u>Maximum Load Rating</u>
PT9020:	650 lbs. (294 kg)
PT9021:	650 lbs. (294 kg)
PT9026:	400 lbs. (181 kg)
PT9027:	400 lbs. (181 kg)



Important! This User Manual contains important information for the user of the product. All who use this product should review and completely understand its contents. Remember to keep this manual in a place where it is always available to those using the product.

Table of Contents

Product Description.....	1
Safety Instructions.....	2
Definitions and Parts	5
Technical Data.....	7
Operation.....	8
Inspection and Maintenance.....	9
Upholstery Cleaning and Care.....	9
Simple Trouble Shooting.....	10
Service & Warranty.....	10

Safety Instructions

- Before using this equipment make sure you have read and understand this entire manual.
- Explain proper use to all employees. This table should be used only by qualified, trained personnel in a clinical setting.
- Never exceed the table's maximum load rating (refer to Technical Data section for load rating).
- Connect table only to an outlet that is grounded and connected to a circuit with a fused circuit breaker in accordance with the National Electrical Code and local codes and ordinances.
- Use only on dry surfaces. Ensure the tables feet do not slide.
- NOT intended for use with accessories other than those indicated in this manual.
- KEEP ALL CHILDREN AWAY FROM TREATMENT AREA. Powered components present a potential crush hazard for children. Allow only the patient and staff in treatment area.
- Always explain to the patient how to properly get on and off the table. Be sure to assist patient to prevent falls.
- Keep patient's hands positioned on the top side of cushions at all times.
- Keep hands and feet away from moving parts and pinch points.
- Ensure controls have locked properly prior to positioning or releasing patient.
- Never leave patient unattended on table.
- Follow preventative maintenance instructions provided in this manual.
- If you need service assistance contact Pivotal Health Solutions at 1-800-743-7738.



Never leave patient on table unattended.



This table is not for patient transport. Casters must remain locked at all times while patient is on table.



Under no circumstances should the table be modified from its original design. Contact Pivotal Health Solutions if you need help with service.



Powered components used to elevate and lower this table present a potential crush hazard. **KEEP CHILDREN AWAY FROM TREATMENT AREA.** Allow only the patient and authorized staff in treatment area.



Keep hands away from pinch points at all times.



ELECTRICAL CONNECTION: For your personal safety, this table must be grounded. This table is equipped with a power supply cord having a three (3) prong grounding plug. To minimize possible shock hazard, it must be plugged into a mating three (3) prong grounding type wall receptacles, grounded in accordance with the National Electrical Code and local coded and ordinances. If a mating wall receptacle is not available, it is the responsibility and obligation of the customer to have a properly grounded three (3) prong wall receptacle installed by a qualified electrician. Power requirements of the table are found in the Technical Data Section.

Warning: Crush Hazard








Keep children away from treatment area.




Stay clear from table when lowering. Only patient and authorized staff should be in treatment area. Risk of serious injury or death if table is lowered on to a child.

Symbols used throughout this manual and on the table indicate potential safety hazards. Be sure you understand these symbols and their definitions before operating the table:



Symbol	Definition
	GENERAL WARNING SIGN: Text with this indicator will explain safety infractions that have the potential to cause serious injury.
	PINCH POINT: Indicates a space on the table with moving parts where a body part could become caught.
	WARNING: DANGEROUS VOLTAGE
	POTENTIAL CRUSH HAZARD: Indicates the presence of a crush hazard from lowering the table.
	Contains electrical equipment. Discard in compliance with local waste regulations.

Definitions & Parts

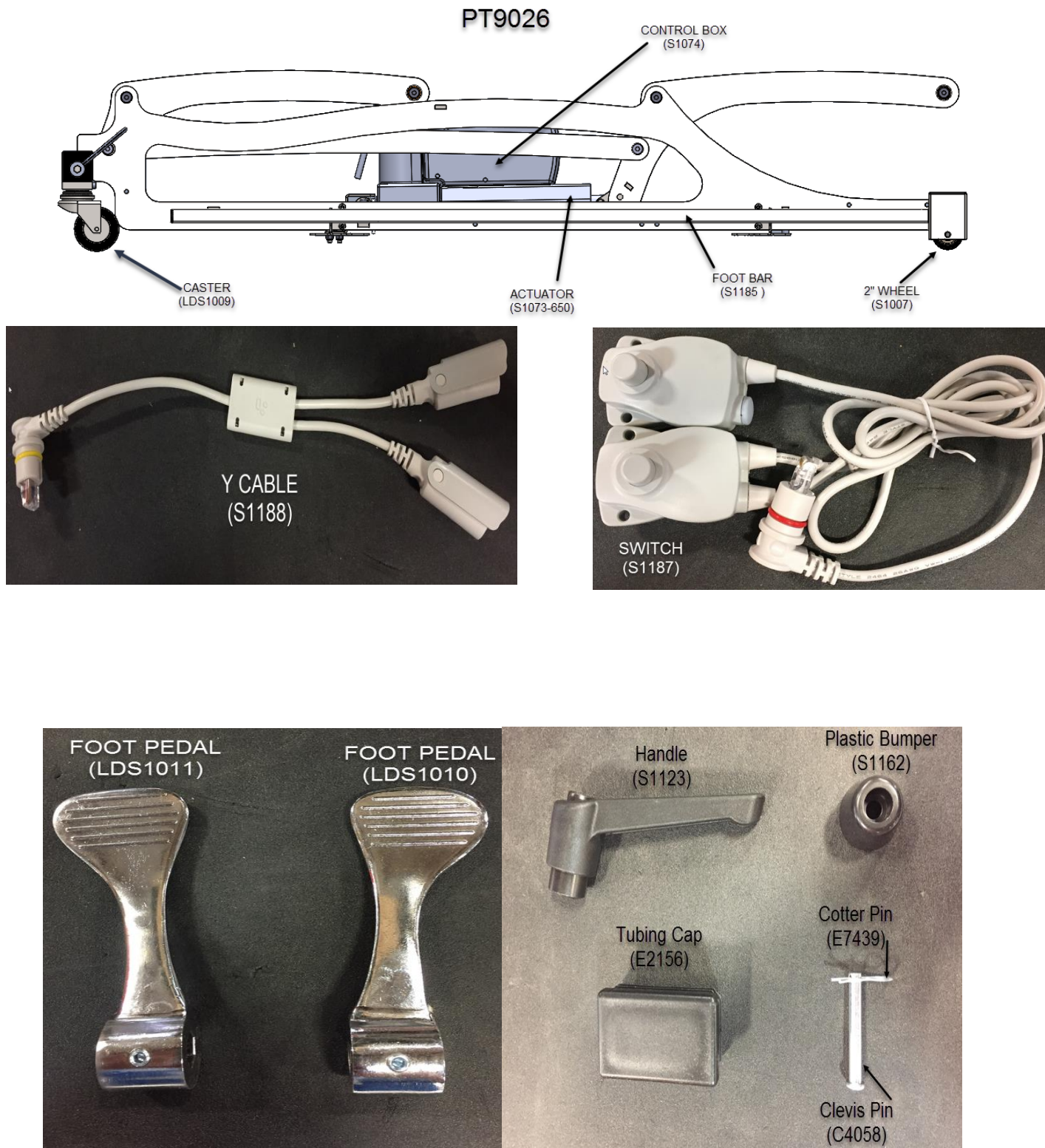



Technical Data

Model Number	PT9020/PT9021	PT9026/PT9027
Input	100-240VAC, 2500mA	100-240VAC, 2500mA
Rated Frequency	50/60 Hz	50/60 Hz
Output	32V	32V
Fuse Type	No user serviceable fuse	No user serviceable fuse
Lifting Capacity	650 lbs. (294 kg)	400 lbs. (181 kg)
Duty Cycle	10% 2 min on 18 min off	10% 2 min on 18 min off
Electrical Classification	Class 1	Class 1
Electrical Type	 Type B	 Type B
IP Rating	IP54	IP54
Tops Available	Basic, Lift Back, Deluxe	Basic, Lift Back, Deluxe



Keep table out of high moisture areas. For indoor use only.

Operating Conditions: This table should be operated, transported and stored in temperatures between 40° F (4.4° C) and 100° F (37.8° C), with relative humidity ranging from 30% - 80%.

Table Physical Specifications:

- Height 17-35"
- Weight 260 lbs.
- Shipping Weight 320 lbs.
- Shipping Dimensions 90 X 32 X 32
- Load Rating 400 Lbs. (181 kg)



TI MOTION Specifications:

- Duty cycle: 10 % ~ 2 minutes of continuous use at full load, followed by 18 minutes of pause
- Ambient temperature: +10° to +40 °C
- Conforms to AAMI STD ES 60601-1, Certified to CSA STD C22.2 NO 60601- 1

Operation

ASSISTING PATIENT ON TABLE:

- 1) The practitioner or trained staff should always assist the patient on the table. Do not allow the patient to get on the table by him/herself.
- 2) The patient should check clothing for keys, wallet, or any other items that may cause discomfort during treatment or damage upholstery.
- 3) Never leave patient unattended while on the table.
- 4) Make sure patient's hands are properly positioned on top side of cushions.
- 5) Always keep hands and feet way from moving parts.

ASSISTING PATIENT OFF TABLE (AFTER TREATMENT):

- 1) Caution the patient not to get up from the table alone so as to prevent any strain that could void the treatment, and to ensure the patient is not lightheaded from laying down which can cause imbalance.
- 2) Allow patient to get full balance before coming to a standing position. Best practice is to keep a hand on the patient at all times.

LOCK CASTERS: To lock casters, place foot on top of CASTER LOCK FOOTPEDAL and apply pressure downward until casters lock. To unlock casters, place foot under CASTER LOCK FOOTPEDAL and lift up.

TABLE ELEVATION (FOOTBAR OPERATION): Use the ELEVATION FOOTBAR to adjust table to desired height. Press down on the foot bar to raise the table. Lift up on the foot bar with the toe of your shoe to lower the table.

TILT (IF EQUIPPED): TILT RELEASE LEVERS are located on each side of the table. These levers release the lock mechanism to allow the cushion to be tilted downward. The TILT RELEASE LEVER is not needed when adjusting the cushion upward because the cushion will automatically lock.

To *Increase* tilt: Lift cushion upward. Position will lock automatically.

To *Decrease* tilt: Support cushion with one hand while twisting the TILT RELEASE LEVER with the other hand to unlock the cushion. Lower to desired tilt. Release TILT RELEASE LEVER to lock.



Never adjust tilt angle while patient's weight is on cushion. Always support the cushion before unlocking.

Inspection and Maintenance

Perform the following checks before each use. If any components fail to operate properly, take THE table out of service until repaired.

- Check for loose hardware. Tighten as needed. Hardware that does tighten needs replacement.
- Inspect table for obvious signs of damage or wear such as cracked welds, loose bolts, frayed or damaged cords.
- Check all moveable components (electrical and non-electrical) to ensure functionality.
- Ensure all locking components lock properly.
- Note: This table requires no lubrication.

Upholstery Cleaning and Care

FOR DAY TO DAY CLEANING: A solution of 10% mild household liquid dish soap with warm water, applied with soft damp cloth. Rinse with clean water and dry.

- **FOR STAINS:** Dampen a soft white cloth in a one to one (1:1) solution of Fantastik® and water OR Formula 409® and water. Rub gently and rinse with a water dampened cloth.







- **FOR MORE DIFFICULT STAINS:** Dampen a soft white cloth with a solution of household bleach (10% bleach / 90% water). Rub gently and rinse with a water dampened cloth to remove bleach concentration. There is a disinfecting wipe on the market called *Protex Ultra Disinfectant Wipes*. Many of our customers use these.

WHAT NOT TO USE: Using the wrong cleaning agents you will VOID YOUR WARRANTY and crack, dry out and destroy your vinyl. Do not use any cleaning agents that contain alcohol, harsh chemicals or abrasives.

Simple Troubleshooting

Table will not raise or lower	<ol style="list-style-type: none">1. Verify table is connected power2. Check to see if foot bar is making contact with switch3. Look for damage to power cord or controls4. Verify outlet is working properly5. <i>If problem persists, contact Pivotal Health Solutions.</i>
If you hear unusual noise	<ol style="list-style-type: none">1. Contact Pivotal Health Solutions.

Service & Warranty

 Pivotal Health Solutions, Inc. 3003 9 th Ave. SW Watertown, SD 57201 USA	 (in US) 800-743-7738  (international) 011-1-605-753-0110  Fax: 605-882-8398  Email: service@pivotalhealthsolutions.com  World Wide Web: www.pivotalhealthsolutions.com
---	--

Additional information:

Expected Life: The product has an approximate expected life time of 10 years when correctly handled, serviced and inspected in accordance with these instructions.

Product changes: Pivotal Health Solutions, Inc. reserves the right to make product changes without prior notice. Contact your Pivotal Health Solutions representative for advice and information about product upgrades.



Rev C 10/11/18