



# Thera-P Traction Table User Manual

Applies to the following Configurations:

**MD**

TXT-1002 TXT-1004

**REF**

TXT-1003 TXT-1006



**Important!** This User Manual contains important information for the user of the product. All who use this product should review and completely understand its contents. Remember to keep this manual in a place where it is always available to those using the product.

## Table of Contents:

Introduction: .....	2
Safety Precautions & Definitions: .....	3
Symbols: .....	4
Installation & Unpacking: .....	5
Table Set up: .....	6
Technical Specifications: .....	7
Operation: .....	9
Preventative Maintenance: .....	11
Upholstery Cleaning & Care: .....	11
Service: .....	12
Troubleshooting: .....	12
Replacement Parts: .....	13
Warranty: .....	14
Manufacturer Information: .....	14

## Introduction:

Comprised of multiple top options, the Thera-P Traction Tables use electric power to help with manual adjustment, allowing for ease of use and control of table positioning. For the practitioner, the Thera-P Traction Table offers a 20-37" full height adjustment and section mobility so the patient can be placed in the proper position for maximum results with minimum effort. For the patient, comfort and easy access getting on and off the table. Each section is adjustable, providing numerous achievable patient positioning options.

**Intended Use:** The Traction table is intended for use in a clinical environment to support patients during treatment.

## Safety Precautions & Definitions:



**WARNING:** Never leave patient on table unattended.



**CRUSH HAZARD:** Powered components used to elevate and lower this table present a potential crush hazard. KEEP CHILDREN AWAY FROM TREATMENT AREA. Allow only the patient and authorized staff in treatment area.













**ELECTRICAL CONNECTION:** For your personal safety, this table must be grounded. This table is equipped with a power supply cord having a three (3) prong grounding plug. To minimize possible shock hazard, it must be plugged into a mating three (3) prong grounding type wall receptacles, grounded in accordance with the National Electrical Code and local codes and ordinances. If a mating wall receptacle is not available, it is the responsibility and obligation of the customer to have a properly grounded three (3) prong wall receptacle installed by a qualified electrician. Power requirements of the table are found in the Technical Data Section.

### Before using this equipment follow the following safety precautions:

- Before using this equipment make sure you have read and understand this entire manual.
- A doctor or clinician authorized by the state must ensure that the patient's health and physical condition are appropriate for any chiropractic treatment being applied when the patient is on the table and treatment must be performed under the direction of a licensed clinician.
- Never exceed the table's maximum supported weight.
- Connect table only to an outlet that is grounded and connected to a circuit with a fused circuit breaker in accordance with the National Electrical Code and local codes and ordinances.
- Use only on dry surfaces to ensure the table's feet or casters do not slide.
- Make sure casters are locked prior to use.
- This table is NOT for patient transport.
- NOT intended for use with accessories other than those indicated in this manual.
- Do not operate the table in an environment where other devices that intentionally radiate electromagnetic energy in an unshielded manner are present.
- Always explain to the patient how to properly get on and off the table. Be sure to assist patient to prevent falls.
- Keep patient's hands positioned on the top side of cushions at all times.
- Keep hands and feet away from moving parts and pinch points.
- Ensure controls have locked properly prior to positioning or releasing patient.
- Never leave patient unattended on table.
- Follow preventative maintenance instructions provided in this manual.
- Ensure all components have been inspected for damage and are fully functional.
- Inspect cables and connectors before each use.
- If you need service assistance contact Pivotal Health Solutions at 1-800-743-7738.

## Symbols:

**Warning and Precautions** through this manual are indicated by specific symbols. Applicable symbols are shown below along with a description of the symbol. Review these symbols and all safety precautions before operating the table.

	<b>CAUTION:</b> Text with a “CAUTION” indicator will explain possible safety infractions that could have the potential to cause injury.
	<b>WARNING:</b> Text with “WARNING” indicator will explain possible safety infractions that will potentially cause serious injury and equipment damage.
	<b>Pinch Point:</b> Indicates space between manually adjustable moving and stationary parts where body parts may become caught, leading to minor injury.
	<b>Warning: High Voltage</b>
	<b>Crush Hazard:</b> Indicates space between powered moving and stationary parts that represent a potential crush hazard.
 	<b>Instructions for Use:</b> Indicates the need for the user to consult the instructions for use.
	<b>EXPLOSION HAZARD</b> Text with “EXPLOSION HAZARD” indicator will explain possible safety infractions if this equipment is used in the presence of flammable anesthetics.
	Contains electronic materials. Recycle and dispose of device properly in accordance with local, state and federal laws.
	Indicates the device manufacturer.



**WARNING:** Under no circumstances should the table be modified from its original design. Contact Pivotal Health Solutions if you need help with service.



**DANGER:** Possible explosion hazard if used in the presence of flammable anesthetics.

## Installation & Unpacking:

**A MINIMUM OF TWO PEOPLE ARE REQUIRED TO LIFT AND MOVE THE TABLE. LIFT THE TABLE BY THE BASE FRAME!**



**WARNING:** Do not install/use this equipment adjacent to, or stacked with, other equipment as it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify they are operating normally.

After removing the top and side panels of the shipping crate, you may notice metal bands and plastic cable ties. These are used to secure various sections of the table and must be removed. Once all bands and cable ties are removed, the table must be lifted directly upward in order to clear all shipping restraints.

Once the table is out of the shipping crate, place table in a level location and inspect the table cushions, frame, cables, etc. for damage. If no damage is noted, plug in the table and test the elevation, drops, and any other options that are on your table. Contact Pivotal Health Solutions or your dealer with any questions or concerns.

**Instructions for Damaged Merchandise:** The contents of this shipment have been checked and packed by experienced personnel. If your company arranged the shipping, damage should be noted on the Bill of Lading before signing to accept the shipment, then articles damaged in transit should be reported at once to delivering carrier and a claim must be filed by you. If freight was arranged by Pivotal Health Solutions, note any damage on the Bill of Lading before signing acceptance of product and notify Pivotal Health Solutions right away. Failure to inspect for shipping damage prior to signing for the product may affect your warranty. We can neither be responsible for, nor can we accept the return of merchandise damaged in transit during shipment arranged by outside parties.






**WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Thera-P Traction Table including cables associated with the Thera-P Traction Table. Otherwise, degradation of the performance of this equipment could result.

## Table Set up:

### Installation of Swing Arm and Traction Unit Mount:

(A minimum of two people required to attach swing arm to table)

1. Unpackage table, swing arm and traction unit mounting plate.
2. Place swing arm on frame mount, insert shoulder bolt into hole in center of the frame mount.
3. A 1/4" allen wrench and 9/16 combination wrench will be needed to attach the nut to the bottom of the shoulder bolt.
4. Lift up on swing arm pin to secure position of swing arm.
5. Place traction unit mount on top of swing arm by sliding mounting plate into the swing arm.
6. Pull pin at top of swing arm to lock the mounting plate in position.

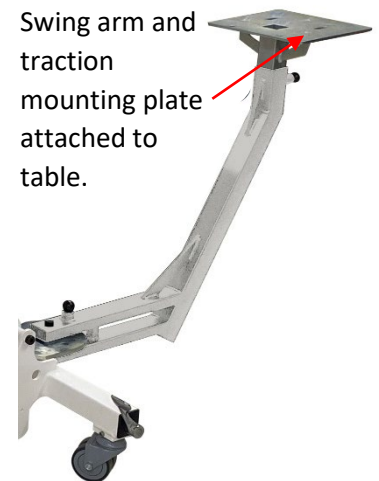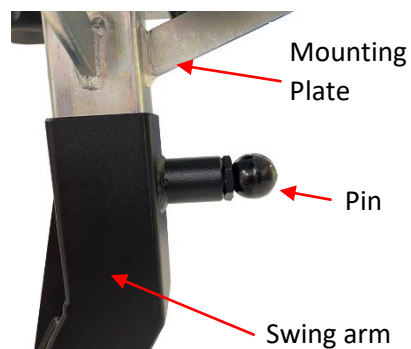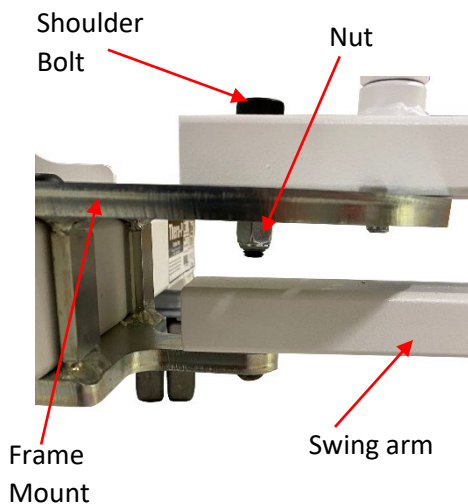


### Mounting Traction Unit to Swing Arm Traction Mount:

1. Place traction unit on traction mounting plate.
2. Align holes on the mounting plate with the holes on the traction unit.
3. Make sure back of traction unit is flush on mounting plate.
4. Locate the knob screws found in hardware pack with the traction unit, secure the traction unit with the enclosed (4) knob screws. Install all 4 knob screws then tighten securely.
5. For operation of a traction unit refer to the manufacture's operating instructions (purchased separately.)



## Technical Specifications:

Base Model Number	TXT
Model Name	Thera-P Traction Table
Cushion Configurations Available	TXT-1002 TXT-1003 TXT-1004 TXT-1006
Dimensions	27" W x 77" L
Height	20-37" H
Head Angle	+30° to -90°
Lift back	0° to 70°
Armrest Horizontal Range	0° to 45°
Armrest Vertical Range	0 to 7"
Foam	2" High Density Foam Cushions
Rated Voltage	120V
Rated Frequency	50/60 Hz
Rated Current	10A
Fuse Type	No User Serviceable Fuse
Safe Working Load	500 lbs. (226.8 kgs)
Duty Cycle	2 Minutes On, 18 Minutes Off
Electrical Classification	Class 1
Electrical Type	 Type B
IP Rating	Foot pedal – IPX1; Other components – see component
Power Cord (120V)	96 in (243 cm) cable length, 10A, unshielded



**WARNING:** Keep table out of high moisture areas. For indoor use only.

**Operating Conditions:** This table should be operated, between 40° F (4.4° C) and 100° F (37.8° C), with relative humidity ranging from 30% - 80%.

**Transport & Storage Conditions:** This table should be transported and stored in temperatures between -40° C (-40° F) and 70° C (158° F) with relative humidity 10%-100%.

**Expected Life:** This product has an approximate expected life time of 10 years when correctly handled, serviced, and inspected in accordance with these instructions.

**Design and Quality:** Pivotal Health Solutions is certified in accordance with ISO 13485, the standard for the medical device industry.

**Product Changes:** Pivotal Health Solutions, Inc. reserves the right to make products changes without prior notice. Contact your Pivotal Health Solutions representative for advice and information about product upgrades.



## Operation:



**WARNING:** Use of accessories and cables other than those specified or provided by Pivotal Health Solutions could result in increased electromagnetic emissions or decreased electromagnetic immunity and result in improper operation.



**WARNING:** Not for patient transport.



**CRUSH HAZARD:** Powered components used to elevate and lower this table present a potential crush hazard. KEEP CHILDREN AWAY FROM TREATMENT AREA. Allow only the patient and authorized staff in treatment area.



**PINCH POINT:** Always keep patient's hands positioned on top side of cushions when lowering, raising and tilting table to keep from potential pinch points.

### Assisting Patient Onto Table:

- 1) Ensure the table is in lowest position and flat configuration prior to start.
- 2) The practitioner or trained staff should always assist the patient on the table. Do not allow the patient to get on the table by him/herself.
  - a. Patient should sit on the center cushion then swing feet up onto the table.
  - b. Patient should not sit on the corners of the table nor the head or foot end.
- 3) The patient should check clothing for keys, wallet, belt, or any other items that may cause discomfort during treatment or damage upholstery.
- 4) Never leave patient unattended while on the table.

### Table Operation:


- 1) To begin treatment, ensure the table is in lowest position and flat.
- 2) Position the patient on the table as above.
- 3) Elevate to the desired height for treatment once the patient is in place.
- 4) Perform the treatment desired. Refer to the 'Operation' section and/or the individual component functions below for operation of specific table features.
- 5) If performing treatment with traction unit:
  - Check swing arm pin to ensure it is secure.
  - Check that the traction unit is installed properly.
  - Follow traction head user manual for operation of the traction unit (purchased separately.)
- 6) After treatment, return the table to a level and straight configuration.
- 7) Lower the table to the lowest position by pressing the foot pedal, or if table is equipped with a foot bar, hold the foot bar up to lower the table.

### Assisting Patient Off Of Table:

- 1) Caution the patient not to get up from the table alone so as to prevent any strain that could void the treatment, and to ensure the patient is not lightheaded from lying down which can cause imbalance.
- 2) Allow patient to get full balance before coming to a standing position. Best practice is to keep a hand on the patient at all times.


### Cushion Movement During Traction Use:

- To lock or unlock cushion during or after treatment or traction used position lever in desired position.

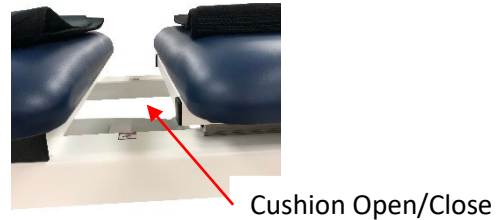

### Adjustment Of The Swing Arm:

- Pull up on knob and pin will release to allow swing arm to move, move swing arm to desired position on frame mount then release knob. Pin will lock swing arm into position when released.

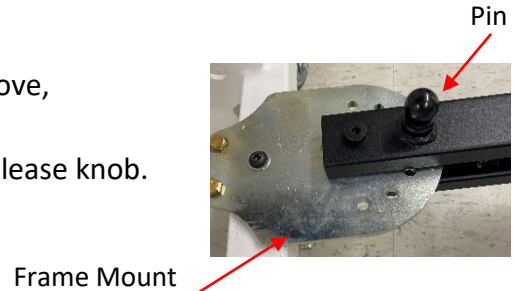

### Elevation:


- Use the Table Height Adjustment foot bar to raise and lower the table height.

### Foot Bar:

- Step on foot bar to raise table. Release at desired position. Lift foot bar with toe of shoe to lower table.

### Traction Strap Frame Attachment:

- Attach traction strap to table frame as shown.

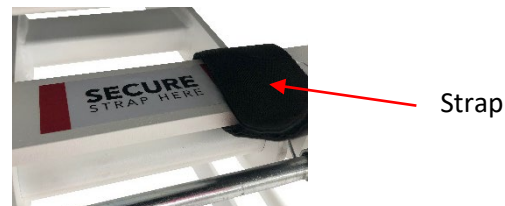

### Tilt Sections: (Various Configurations Optional)

- Lift on cushion to tilt up. Cushion position will lock automatically. To lower, support cushion and twist lock lever to unlock the cushion. Lower the cushion to the desired angle, and release twist lock lever. Make sure to hold cushions by side, not ends to avoid pinch points.



**WARNING:** To prevent accidental disengagement the traction unit must be securely attached to the mounting surface of the traction table. It is the responsibility of the user to verify the adequacy of the installation before use in patient therapy.

## Preventative Maintenance:



**WARNING:** Disconnect table from power source before wiping down mechanical parts to avoid personal injury.

### INSPECTION:

At least monthly the table should be thoroughly inspected for wear and tear, loose hardware and parts, and other damage.

- Inspect the table to make sure that there is no external damage or loose hardware. Tighten loose hardware. Hardware that does not tighten needs replacement.
- Inspect table for obvious signs of damage or wear such as cracked welds, loose bolts, frayed or damaged cords. Do not use a table with damaged cords.
- Check moveable components (electrical and non-electrical) to ensure functionality.
- Check components to ensure they lock and function properly.
- **Contact Pivotal Health Solutions with questions or concerns.**

## Upholstery Cleaning & Care:



**CAUTION:** Read carefully, improper cleaning will void the warranty. Never use alcohol-based cleaning agents.

**DISINFECTING:** There is a disinfecting wipe on the market called *Protex Ultra Disinfectant Wipes*. Many of our customers use these.

**FOR DAY TO DAY CLEANING:** A solution of 10% mild household liquid dish soap with warm water, applied with soft damp cloth. Rinse with clean water and dry.

**FOR STAINS:** Dampen a soft white cloth in a one to one (1:1) solution of Fantastik® and water OR Formula 409® and water. Rub gently and rinse with a water dampened cloth.

**FOR MORE DIFFICULT STAINS:** Dampen a soft white cloth with a solution of household bleach (10% bleach / 90% water). Rub gently and rinse with a water dampened cloth to remove bleach concentration.

**WHAT NOT TO USE:** Using the wrong cleaning agents you will VOID YOUR WARRANTY and crack, dry out and destroy your vinyl. Do not use any cleaning agents that contain alcohol, harsh chemicals or abrasives.

## Service:

**Refer to Warranty section in this manual for specific warranty information.**

When replacing worn parts, use genuine Pivotal Health Solutions, Inc. parts by contacting our service department. When ordering replacement parts:

- Ask for the service department, 800-743-7738
- Have the model number and serial number available
- Specify parts by the numbers/description below

## Troubleshooting:

No Power	<ol style="list-style-type: none"><li>1. Verify power supply connections.</li><li>2. Check that outlet is working and breaker is not tripped.</li><li>3. <i>If the problem persists, contact Pivotal Health Solutions.</i></li></ol>
Elevation: Table will not raise or lower.	<ol style="list-style-type: none"><li>1. Check to verify table is home by lowering all table sections flat in the horizontal direction.</li><li>2. Check to see if cord running from foot control to control box is connected properly.</li><li>3. Look for damage to controls.</li><li>4. Make sure outlet is working properly.</li><li>5. <i>If the problem persists, contact Pivotal Health Solutions.</i></li></ol>
If you hear unusual noise.	<i>Contact Pivotal Health Solutions</i>

### Technical Information:

Pivotal Health Solutions will make available, upon request, circuit diagrams, component part lists, descriptions, calibration instructions, or other information that will assist service personnel.

### Resale or Disposal:

A therapy table is considered a medical device by the Food and Drug Administration. Therefore, it is necessary that Pivotal Health Solutions, Inc. be notified if the table is sold, destroyed, or otherwise disposed of. Please notify in writing providing your name and the serial number of your table to:

**Pivotal Health Solutions**  
**Attn: Quality**  
**3003 9<sup>th</sup> Ave. SW**  
**Watertown, SD 57201**



Recycle and dispose of device properly in accordance with local, state and federal laws. Over the years, tons of electronics equipment with hazardous materials have been thrown away with standard garbage. Over time, these materials leech out of the electronic causing damage to the environment. It is important to try and properly dispose of retired devices in order to prevent damage to our environment.

## Replacement Parts:



NOTE: These tables may have various cushion configurations. Please contact Pivotal Health Solutions for replacement cushion details.

<u>Part Name</u>	<u>Part Number</u>
Face Cushion (27")	S1305-xxx
Chest Cushion (27")	S1308-xxx
Pelvic Cushion (27")	S1311-xxx
Leg Cushion (27")	S1314-xxx
Fixed Lever Black Handle	S1123
Traction Unit Mount	S1356
Swing Arm	S1355
Release Lever	S1339
Ball Knob	S1348
Traction Frame Mount	S1315
Caster Wheel	LDS1009
18" Friction Bar	S1017
13.25" Friction Bar	S1021
2" Wheel	S1007
Knob	88282

\*xxx is a place holder for the color code. Contact Pivotal Health for the appropriate color code for your table.

## Warranty:

**Warranty Period:** <https://www.phsmedicalsolutions.com/customer-service/service-and-warranty>

**Note to user:** If a serious incident occurs, report the incident to Pivotal Health solutions. (See section with Manufacturer Contact Information for contact details). If the incident occurs in the EU, also report any serious incident to the Competent Authority of the Member State in which the user and/or patient is established.

## Manufacturer Information:

Rev F 04-29-2025



**Pivotal Health Solutions, Inc.**  
3003 9<sup>th</sup> Ave. SW  
Watertown, SD 57201  
USA



800-743-7738



011-1-605-753-0110



Fax: 605-882-8398



[service@pivotalhealthsolutions.com](mailto:service@pivotalhealthsolutions.com)



[www.pivotalhealthsolutions.com](http://www.pivotalhealthsolutions.com)



Emergo Europe  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands



Sissel France SAS  
14 rue des Grandes Bosses  
44220 Couron  
France



UK Responsible Person  
Emergo Consulting (UK) Limited  
Compass House, Vision Park Histon  
Cambridge CB24 9BZ  
United Kingdom



Performance Health International Ltd.  
Nunn Brook Rd, Huthwaite Sutton-Ashfield,  
Nottinghamshire NG17 2HU  
United Kingdom



MedEnvoy  
Gotthardstrasse 28  
6302 Zug  
Switzerland



Medidor AG  
Lenzburgerstrasse 2  
CH-5702 Niederlenz  
Switzerland