

Who We Are

First impressions mean a lot.

Here at The Athletic Edge, we understand that. It's why we put such care into working with each of our customers to ensure that their training rooms, locker areas and general facilities not only meet their specific needs, but look great too. Our custom products tell a story—your story.

We promise that each product meets our exacting standards in terms of durability, user-friendliness, patient comfort and brand-ability. Our products are built by skilled craftsmen and women in our Watertown, SD facility and in our second location in Springfield, OR.

We have the EDGE. Do you?

Contact Us

Pivotal Health Solutions
DBA: The Athletic Edge
3003 9th Ave. SW
Watertown, SD 57201

Phone: (800) 743-7738

Email: info@pivotalhealthsolutions.com

Web: www.teamedgeathletics.com



TREATMENT TABLE & CABINET GUIDE



Aluma Elite

Pivotal Health Solutions
DBA: The Athletic Edge
3003 9th Ave. SW
Watertown, SD 57201

Table of Contents

A4201/A4201-PRO	1
A6000/A6000-EL.....	2
A6000/A6000-EL Electrical Specifications	3
Upholstery Cleaning & Care	4
Warranty	5

Unpacking

A minimum of two people are required to lift and move the table. Lift the table by the base frame.

After removing the top and side panels of the shipping crate, you may notice metal bands and plastic cable ties. These are used to secure various sections of the table and must be removed. Once all bands and cable ties are removed, the table must be lifted directly upward in order to clear all shipping restraints.

Once the table is out of the shipping crate, place table in a level location and inspect the table cushions, frame, cables, etc. for damage. If no damage is noted, plug in the table and test the elevation, drops, and any other options that are on your table. Contact Pivotal Health Solutions or your dealer with any questions or concerns.

Warranty

Pivotal Health Solutions warrants to the original purchaser that each Athletic Edge™ product shall be free from defects in material and workmanship and agrees to remedy any such defect or to furnish a new or equal part in exchange therefore. Warranty covers aluminum frames for 10 years, steel or wood frames for 5 years, laminate, phenolic, or plastic frames for 3 years, electrical components, standard foam and upholstery for 2 years, and moving components, hardware, accessories, and custom-printed upholstery and graphics for the first year, commencing on date of original shipment. Labor coverage may not include replacement or installation of small parts or components. Pivotal Health Solutions Service and/or Sales will determine the technical level of service needed and approve labor coverage on a case by case basis.

Pivotal Health Solutions reserves the right to inspect claimed defective products. No returns, replacements or repairs may be made without prior written consent. Please do not return any product to Pivotal Health Solutions without first contacting Customer Service for a Return Authorization number. Products returned without a Return Authorization number will be refused back to the customer at their expense.

This Warranty covers defects in materials and workmanship provided the product is used for the normal purpose for which intended. The customer shall be responsible for the proper set-up and use of the product. This Warranty does not include damage resulting from accident and expressly excludes normal wearing of parts or deterioration due to normal wear and tear, damage caused by improper set-up or storage, defect caused by transportation, accident, fire, flood, alteration, abuse, misuse, or negligence. Pivotal Health Solutions shall not be liable for any direct or indirect, incidental, exemplary, or consequential damages or delay, including damages for loss of income or loss of use.

Any obligation of Pivotal Health Solutions under this Warranty shall automatically and immediately terminate, without notice from or any further action by Pivotal Health Solutions and Pivotal Health Solutions shall have no responsibility for damages of any kind as a result of the occurrence of any of the following:

- accident, misuse, abuse or negligent use of the Products or any component thereof;
- any repair or alteration of the Products or any component thereof made outside Pivotal Health Solutions' authorization;
- improper installation, setup, or operation (including both mechanical and electrical) of the Products or any component thereof, which includes operation of the Product not in accordance with the Product's operating manual;
- failure to provide normal maintenance for the Products or any component thereof in accordance with the Product Operating Manual.
- alteration or obliteration of any identifying marks.

Upholstery Cleaning & Care

DISINFECTING: There is a disinfecting wipe on the market called *Protex Ultra Disinfectant Wipes*. Many of our customers use these.


FOR DAY TO DAY CLEANING: A solution of 10% mild household liquid dish soap with warm water, applied with soft damp cloth. Rinse with clean water and dry.

FOR STAINS: Dampen a soft white cloth in a one to one (1:1) solution of Fantastik® and water OR Formula 409® and water. Rub gently and rinse with a water dampened cloth.

FOR MORE DIFFICULT STAINS: Dampen a soft white cloth with a solution of household bleach (10% bleach / 90% water). Rub gently and rinse with a water dampened cloth to remove bleach concentration.

WHAT NOT TO USE: Using the wrong cleaning agents you will VOID YOUR WARRANTY and crack, dry out and destroy your vinyl. Do not use any cleaning agents that contain alcohol, harsh chemicals or abrasives.

A4201/A4201-PRO

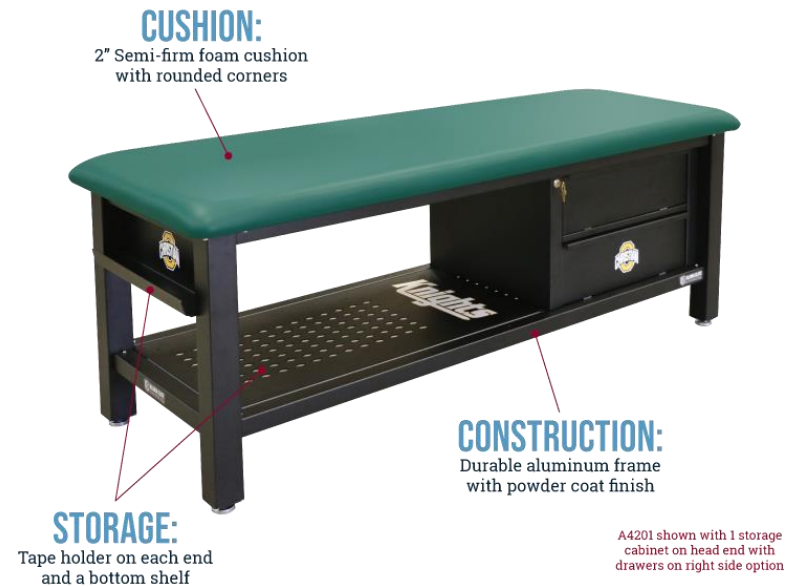


Standard Features

- 750 lbs. Load rating
- Durable aluminum construction
- Tough powder coat finish
- Tape holder on each end and a bottom shelf
- Graphics Package: includes (1) school logo/name decal on each tape holder and (2) on bottom shelf
- 2" Semi-firm foam cushion

Options

- 70° Lift Back Cushion
- 45° Split Leg Cushion
- Storage Cabinet with 7"H Drawers
- Drawer Mats
- Slip Cover
- Decal Package

A6000/A6000-EL

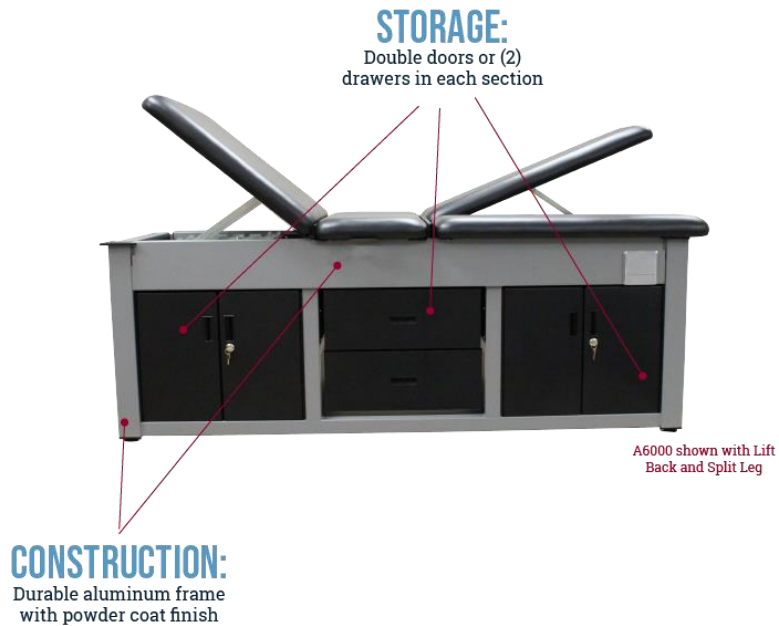

Standard Features

- 750 lbs. Load rating
- Durable aluminum construction
- Tough powder coat finish
- Pass through storage accessible on both sides
- USB/120V Outlet on each side
- Graphics Package: includes (1) large logo decal on foot end

Options

- 70° Lift Back Cushion
- 45° Split Leg Cushion
- 31"-41" Elevation with Single Mounted Hand Control
- Heavy Duty Drawer Liner
- Slip Cover
- Decal Package

A6000/A6000-EL Electrical Specifications



Model Number	A6000	A6000-EL
Medical Grade Power Strip?	YES	YES
Elevation?	NO	YES
Rated Voltage	120V	120V
Rated Frequency	60Hz	60Hz
Rated Current	20A	20A
Outlet Voltage	120V	120V
Outlet Frequency	60Hz	60Hz
USB Voltage	5VDC	5VDC